



Spirobank II Basic Quick Start

1. The Spirobank II Basic can be turned on or off by holding this button.

There will be 3 screens that appear:
The first is the MIR logo splash screen.
This shows the date & time



MIR USA, Inc.


1900 Pewaukee Road, Suite D - Waukesha, WI 53188
P +1 (262) 565 - 6797 F +1 (262) 364 - 2030
mirusa@spirometry.com www.spirometry.com

MIR ITALY, Head Office

Via del Magliolino, 125 - 00155 Roma
P +39 06 22 754 777 F +39 06 22 754 785
mir@spirometry.com www.spirometry.com

The second screen shows

the number of patient files,
the number of Spirometry tests,
the amount of free memory
and the battery symbol
indicating amount of battery charge.

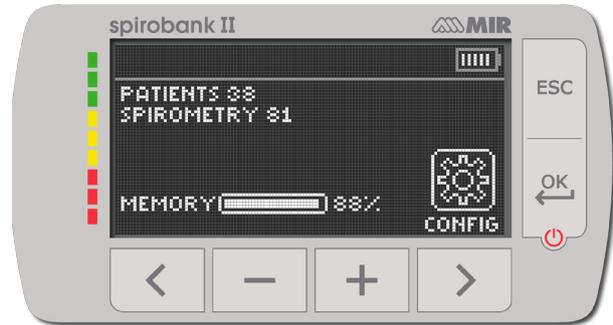

It will also show you the configuration icon that brings you to the **Configuration Menu**.



Pressing the Right Arrow
under this symbol will take you to the
Configuration Menu.

**The third screen is the Main Screen
where you can choose a function**

You can also navigate the
configuration menu from the main screen
pressing the **ESC button**



Use the left & right arrow buttons to scroll up
and down through configuration menu

Entering patient information

Press the left arrow
under the I.D. symbol



This screen appears:



There are 2 choices for entering patient information


Choose New by pressing the left arrow
to create a new patient.

Choose Modify by pressing the right arrow
to edit an existing patient file.

Selecting New patient: the insert patient data screen will appear.

Use the minus and plus buttons to increase/decrease
numbers in the number fields.

Once birthdate, height (inches or cm), and weight
(lb or kg) have been entered 2 choices will appear
for gender press the minus to highlight male or
press the plus sign to highlight female.

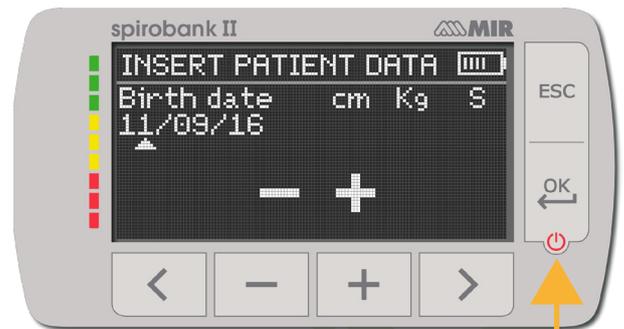

Then press the **ok button** to make your choice
Right after that the ethnicity options will appear
Use the left and right arrow to scroll up and down through the choices
press the ok button to confirm your choice.

Now you will be back on the main screen

To make an FVC test

Insert the FlowMIR turbine with the cardboard mouthpiece pointing out the same side that screen is on and twist it clockwise to lock

Back on the main screen.


Press the minus sign under the spiro symbol



A new screen appears with spirometry test choices.

(The POST choice will be grayed out until at least one Pre FVC test is done).

Press the button under the choice of test.

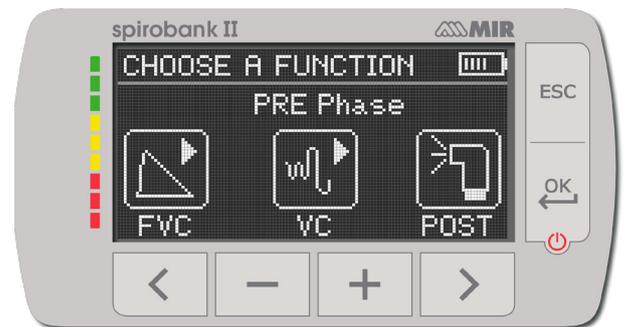

The next screen will have instructions about performing the test.

The patient may begin to perform the test at his screen.

The test will time out if not started in time

Connect the device to the computer via the micro USB cable to charge it.

The screen will say 'external control' and will remain on while connected.



MIR USA, Inc.

1900 Pewaukee Road, Suite D - Waukesha, WI 53188

P +1 (262) 565 - 6797 F +1 (262) 364 - 2030

mirusa@spirometry.com www.spirometry.com

MIR ITALY, Head Office

Via del Maggiolino, 125 - 00155 Roma

P +39 06 22 754 777 F +39 06 22 754 785

mir@spirometry.com www.spirometry.com

-  To access the default configuration menu
-  To access patient data from the main display
-  To perform a new test of a patient recalled from the patient records
-  To insert new patient data
-  To modify patient data
-  To display the most recent test of a patient
-  To show the last test performed
-  To access the database of the performed tests
-  To search a test with the date of birth of a patient
-  To search a test starting from a specific date onwards (partial database)
-  To flick through a database from beginning to end and viceversa (complete database)
-  Male patient selection
-  Female patient selection
-  To access spirometry testing type
-  To perform a forced vital capacity spirometry test FVC/search FVC tests in memory
-  To perform a slow vital capacity spirometry test VC/search VC tests in memory
-  To perform a spirometry test with a broncodilator

MIR USA, Inc.

1900 Pewaukee Road, Suite D - Waukesha, WI 53188
P +1 (262) 565 - 6797 F +1 (262) 364 - 2030
mirusa@spirometry.com www.spirometry.com

MIR ITALY, Head Office

Via del Maggiolino, 125 - 00155 Roma
P +39 06 22 754 777 F +39 06 22 754 785
mir@spirometry.com www.spirometry.com