

BioSecure



+ INTUITIVE

+ ADAPTABLE

+ COMPACT

+ INNOVATIVE

BioSecure



2L



3L



5L



7L



9L



12L

BioSecure




1. Ambidextrous handles
2. Closure system
3. Aperture for receiving waste material
4. Open flap's anchoring point
5. Anchoring point for the temporary closure feature
6. Liquid-retaining slope to prevent accidental leaks

7. Sharps removal tabs
8. Flap handling area
9. Main lid
10. Flap release hinge
11. Tabs for final closure feature
12. Flap

BioSecure




Press down the corners; press down the sides. Audible clicks confirm the lid and base are assembled correctly

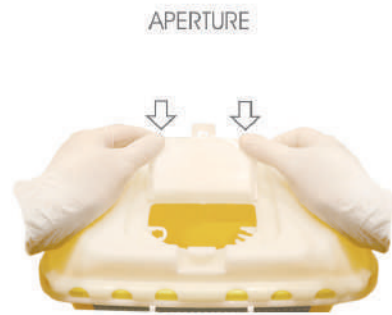

The lid includes a hinge for retention of the flap (open when in use). Handle the flap using the side areas. Press down the hinge to release the flap



To engage the temporary closure feature release lid from the open position pressing down the hinge. Place the flap so that it covers the aperture. Press down the side areas of the flap and continue until a click is heard

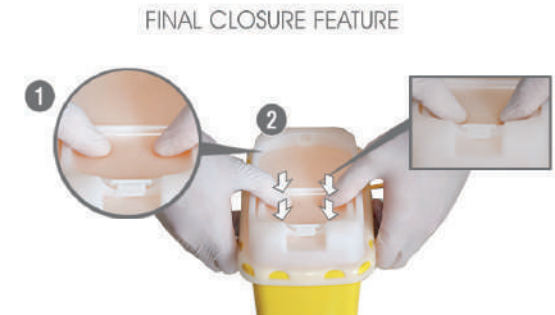

To engage the final closure feature, place the lid in the temporary closure position. Press down on the circles drawn on the tab; several clicks will be heard



Sharps removers suitable for Vacutainer, insulin pen needles, blades, etc.



Carry handle
Handle enables the use of a wall bracket




Bracket suitable for trolleys, horizontal/vertical rods and walls


BioSecure



SHARPS REMOVAL TABS

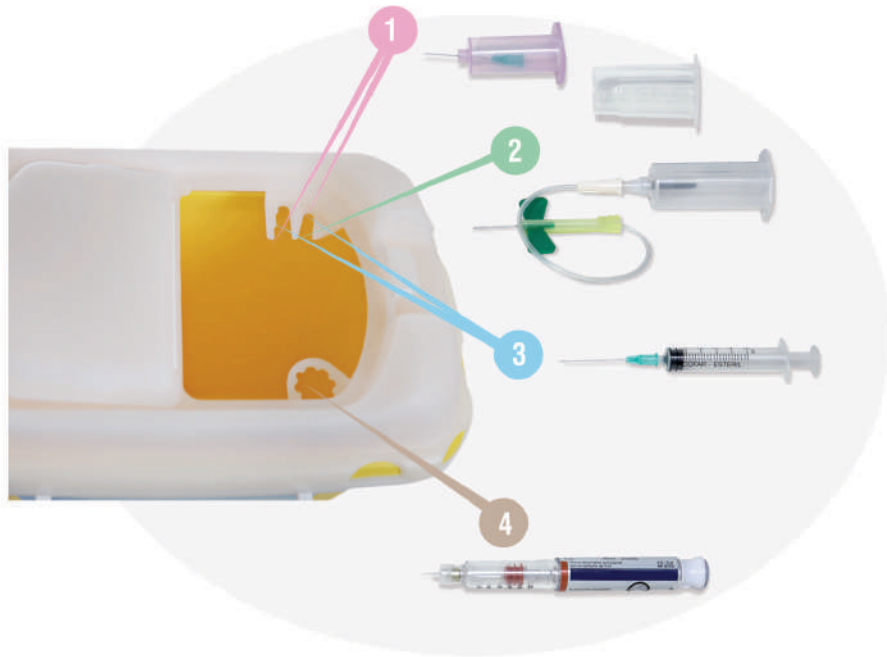

1 BLOOD COLLECTION SYSTEMS

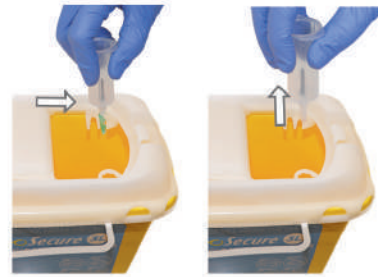

2 WINGED BLOOD COLLECTION (BUTTERFLY)



3 LUER AND LUER LOCK



4 INSULIN



BioSecure



FILL LINE



FILL LINE
Fill line clearly visible externally

- ⚠ NEVER FORCE THE INSERTION OF WASTE
- ⚠ 80% OF THE USABLE CAPACITY NOT TO EXCEED



TECHNICAL DATA

ORDER CODE	NOMINAL CAPACITY (l)	REAL CAPACITY (l)	WEIGHT (kg)	WIDTH (cm)	LENGTH (cm)	HEIGHT (cm)	MAXIMUM GROSS WEIGHT (kg)
BSC02L	2	1.97	0.15	11.8	19.8	13.06	1.13
BSC03L	3	2.71	0.18	11.8	19.8	18.56	1.47
BSC05L	5	4.12	0.23	19.3	19.3	18.6	3.0
BSC07L	7	6.07	0.31	19.3	19.3	29.4	3.2
BSC09L	9	8.03	0.36	18.8	29.7	21	4.0
BSC12L	12	10.96	0.45	18.8	29.7	29	5.0

ORDER CODE	NOMINAL CAPACITY (l)	REAL CAPACITY (l)	WEIGHT (kg)	WIDTH (cm)	LENGTH (cm)	HEIGHT (cm)	MAXIMUM GROSS WEIGHT (kg)
TAPA BSC T2-3L		NA	0.075	12.25	20.25	3.96	NA
TAPA BSC T5- 7L	NA	NA	0.105	19.87	19.87	3.87	NA
TAPA BSC T9-12L	NA	NA	0.155	20.05	30.35	3.87	NA
SPBABSC			0.06	15.15	0.06	NA	

BioSecure



LOGISTICS

PRODUCT	ORDER CODE	CARTON QTY.	CARTON DIMENSIONS(cm)	CARTON WEIGHT(kg)	PALLET QTY.	PALLET SIZE	PALLET DIMENSIONS(cm)	PALLET WEIGHT(kg)	FULL PALLET WEIGHT(kg)
Biosecure 2L	BSC02L	70	40 x 40 x 70	15.40	1260	EURO	120 x 80 x 231	15	293
Biosecure 3L	BSC03L	65	40 x 40 x 70	17.30	1170	EURO	120 x 80 x 226	15	326
Biosecure 5L	BSC05L	60	40 x 40 x 72	21.00	1080	EURO	120 x 80 x 233	15	393
Biosecure 7L	BSC07L	45	40 x 40 x 67	19.60	810	EURO	120 x 80 x 225	15	368
Biosecure 9L	BSC09L	45	60 x 40 x 70	23.17	540	EURO	120 x 80 x 225	15	294
Biosecure 12L	BSC012L	35	60 x 40 x 70	21.10	420	EURO	120 x 80 x 225	15	268

BioSecure

Sanypick S.A

C/ Carlos Jiménez Díaz, 23. ES-28806 Alcalá de Henares (Madrid)
+34 91 884 1 86 · admon@sanypick.com
www.sanypick.com