USER MANUAL

SCALEO
M E D I C A L

Active transfer slings

CE

Content

WARNING AND SAFETY INSTRUCTIONS	3
SYMBOLS AND PICTOGRAMS.....	4
GENERAL INFORMATION	5
INTENDED USE	5
CONTRAINDICATIONS	6
INSTRUCTIONS FOR USE	7
RAISING AND TRANSFERRING THE PATIENT	7
CHOICE OF SLING	9
RANGE OF SLINGS	10
SLINGS FOR QUICK ACTIVE TRANSFER	10
CLEANING, DISINFECTING AND MAINTENANCE.....	11
CLEANING AND DISINFECTING	11
PERIODIC INSPECTION AND PREVENTIVE MAINTENANCE.....	11
TECHNICAL SPECIFICATIONS	12
WARRANTY, LIABILITY AND END OF LIFE.....	13
WARRANTY	13
LIABILITY	13
END OF LIFE	13
EU DECLARATION OF CONFORMITY	14
CONTACT INFORMATION.....	19

Warning and safety instructions

This user manual contains important safety instructions and information concerning the use of the SCALEO Medical active transfer slings. Carefully read the manual before using them with the lifters.

Before performing a transfer, make sure the sling is adapted to the pathology and corpulence of the patient, and in particular that the patient weight does not exceed the maximum capacity of the sling and/or of the patient lifter you are using. A sudden break up of an element could cause severe injury to the patient and/or the user.

Torn, cut, frayed or broken slings can fail, resulting in serious injury to the patient and/or to the user. Only use slings in good condition.

The slings and the lifter are intended to lift and transfer people. Never use them to lift or to carry objects.

Slings should never be bleached. The washing conditions and temperatures are showed on the label.

The procedure for positioning the patient on the hoist is available in the user manual of the patient lifter. Please refer to the manual for details.

Symbols and pictograms

Symbols used in this manual:

This symbol is used to point out instructions and information related to safety in the working environment, where injury may occur if the information is disregarded or ignored. Follow these instructions and be careful and attentive at all times.

This symbol indicates important information regarding the use of the equipment. If not taken into consideration, it may lead to damage or functional defects to the equipment.

This symbol indicates important and useful information. If taken into consideration, it will help the operator to work efficiently. It may help simplify routines and to explain complicated facts.

Symbols used in labels:

Symbol	Meaning
	Manufacturer
	Warning: read the user manual
	Read user manual for instruction
	In accordance with applicable European regulations
	Only wipe
	Do not put in washing machine

Symbol	Meaning
	Machine washable
	Do not tumble dry
	Do not bleach
	Do not iron
	Medical Device

General information

SCALEO Medical active slings are Class I medical devices, according to the European medical device regulation 2017/745. They are compliant with the EN 10535 standard.

This user manual concerns the following slings:

- Quick transfer slings for home care (S8594XXX),
- Quick transfer Cocoon slings with abdominal belt (S8591xxx),
- Self-adjusting slings with abdominal belt (S8590xxx),
- Comfort slings with abdominal Belt (S8592xxx).

The xxx of the reference represents the 3 digits that correspond to the size of the sling.

They ensure a perfect back support and are easy to install in a few seconds thanks to its exclusive attachment system.

This exclusive ergonomic transfer concept avoids the traumatic vertical traction perceived on first-generation stand to sit lifters and help restoring the natural movement of the body.

The abdominal belt limits the sliding effect and reassures the transferred person.

Intended use

Designed for 2-point handlebars, the SCALEO active slings are intended to be used in combination with a stand to sit patient lifter to perform sitting to sitting transfers.

They are compatible with the following equipment:

- Poweo® Activ'
- Poweo® City

Contraindications

The active transfer slings of SCALEO Medical are NOT intended to be used:

- When a patient is subject to loss of balance or discomfort,
- For patients whose mental state is impaired (agitated patients)
- In the case of medical contraindication of support following surgery of the shoulder, hip, knee or ankle,
- In patients with severe flexion deformity of the knee amputation not allowing a standing position, even with support,
- In disabled patients of the lower limbs,
- In patients with curvature of the spine,
- In patients with muscle spasm,
- In patients weighing more than the maximum load of the sling,
- To transfer patients over a long distance,
- To transfer patients on slopes.

Instructions for use

Raising and transferring the patient

For an ergonomic transfer, it is recommended to maintain a 90° angle for the back, the knees and the ankles.

If the patient to be transferred is on a wheelchair, do not forget to block the brakes of the chair before any transfer procedure.



Should the patient have its gravity center offset due to a very large size or morphology, it is highly recommended to compensate this offset by shortening one of the sling cords and by avoiding to raise the patient in a full upright position.

Whatever stand aid model is used, assisted rapid transfer is only for patients with minimal tone in the lower limbs. When transferring, it is important that the patient's shins are in contact with the lifter.

1. Place the sling on the lower back of the patient, making sure the bottom edge of the sling is at the base of the spine. Pass the belt under the arms of the patient.
2. Secure the belt with the buckle and tighten the belt to secure the installation of the sling.
3. Lower the handlebar to a minimum height while keeping it raised. Brake the lifter for agitated patients or if the patient has spasms.
4. Position the lifter facing the patient, open the legs of the lifter if needed. The feet of the patient should rest on the base of the lifter.
5. Lower the handle, then pass the sling under the arms of the patient.
6. Secure the sling cords tightly and symmetrically, by positioning them in the notch on the handlebars. The ropes should preferably be placed from the inside to the outside of the

harness latch notch. It must be firmly tightened and locked according to the following drawings:

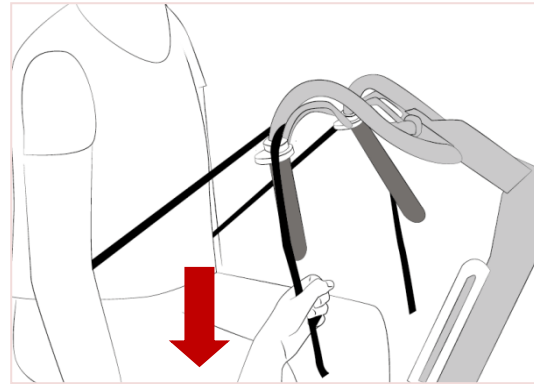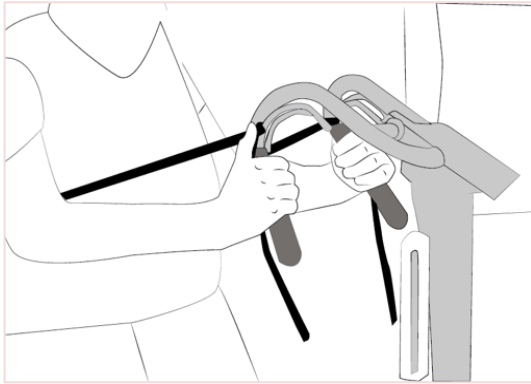“Stand” down the open to secure the installation.
hook.

The cords must be tightened firmly before lifting. They will slide and lock under the effect of the weight during lifting. If the cords are not tight enough, lifting could be asymmetrical and less comfortable.

If the patient is heavy, it is recommended to attach the cords to the outside of the handlebars and increase their amplitude. This will avoid pinching.

7. Using the remote control, gently lift the patient and tighten the cords until his buttocks are off the chair.
8. Unlock the brakes of the lifter and transfer the patient.

Ensure that the patient holds the handlebar with his hands during the transfer, or at least that his hands are out of reach of any moving part of the lifter.

Avoid raising the patient to a full standing position; this position is less comfortable than the sitting position in the transfer operation.

Choice of sling

The recommended slings for the Poweo® Activ' and Poweo® City are the following:

Reminder: it is essential to use the correct type of sling for the patient's condition, weight and height. It is also important to choose the appropriate sling for the type of transfer and the environment.

Size	Patient weight		Waist size
	kg	lbs.	cm
XS	20-30	44-66	50-65
S	30-50	66-110	60-90
M	50-90	110-198	75-110
L	90-160	198-352	100-150
XL	160-200	352-440	120-180

SCALEO Medical offers a range of sling models to suit different contexts of use and user needs. The institutional slings are equipped with a label to record the periodic inspections required by EN 10535.

Range of slings

Slings for quick active transfer

Polyester active transfer sling Cocoon

The Cocoon slings feature a polyurethane coated polyester interior lining. This coating eliminates the gliding effect, provides more flexibility and a softness sensation.

Polyester active transfer sling for home

They also offer a polyurethane coated polyester interior lining.

Polyester quick active self adjusting sling

The self-adjusting function of the slings is naturally activated during the standing process, thanks to the abdominal retraction of the patient.

Polyurethane active Comfort slings

The particularly generous padding of the sub-axillary parts of the Comfort sling provides a maximum comfort and avoids shearing. Its polyurethane design provides a good elasticity and its impermeability limits the risks of nosocomial infections.

Size	Polyester active transfer sling for home	Polyester active transfer sling Cocoon	Polyester quick active self adjusting sling	Polyurethan active Comfort sling
S	S8594100	S8591100	S8590100	S8592100
M	S8594200	S8591200	S8590200	S8592200
L	S8594300	S8591300	S8590300	S8592300
XL	S8594400	S8591400	S8590400	S8592400

Cleaning, disinfecting and maintenance

Cleaning and disinfecting

In order to prevent nosocomial infections, prefer vinyl or polyester slings. In the case of severe risk, discard the old sling and use a new one.

Good cleaning and maintenance practices allow a longer life and upmost security of the slings.

- Slings must be washed separately, allowing a cleaning without machine drying with standard cleaning products.
- The washing temperature and the allowed products for cleaning are indicated on the label.
- Never use autoclaves.
- Air dry or on a flat surface, never use a dryer.

The polyurethane and polyester slings are “softened” when dried or when washed in washing tunnels. This reduces their efficiency and their lifespan as it may cause serious shear to the sling.

Periodic inspection and preventive maintenance

The preventive maintenance for the slings must be performed regularly every six (6) months, according to annex B of the EN ISO 10535 standard and the recommendations from the competent health authorities.

1. Check the readability of the label.
2. Check carefully for wear and damage to the following:
 - Fabric,
 - Straps,
 - Seams,
 - Buckles.

Always verify the status of the sling before use and especially after washing it. If a single point of control is inconsistent DO NOT use the sling. A damaged sling breaking can cause serious injury to the patient.

It is strongly advised to keep all inspection records in the event of an incident.

SCALEO Medical should be informed of the result of inspections performed on its devices.

A traceability label on the back of the sling enables to keep track of the preventive controls and the cleaning and disinfecting operations. Use a permanent marker to record your intervention.

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Technical specifications

The life time of the sling depends mainly on the frequency of use and of the cleaning protocol. As a guide a sling used daily and maintained weekly in accordance with the requirements can normally be used during five years.

Medical devices class	Class I, according to the European Medical device Regulation 2017/745
Materials	Polyester, polyurethane.
Compliant to standards	Compliant with EN 10535, EN ISO 12952-1 and EN ISO 12952-2 standards

Warranty, liability and end of life

Warranty

The active transfer slings of SCALEO Medical have a one (1) year warranty.

- Any abnormal use or non-compliant usage will result in the cancellation of the warranty.
- Any modifications to the product will render the warranty void.

Liability

SCALEO Medical assumes no liability for any injury or harm and consequences thereof directly or indirectly caused to operators, patients, or any third party in the following cases:

- Non-compliance of the instructions and recommendations supplied within the present user manual.
- Abnormal use of the equipment, negligence, accident, human error, or maintenance and cleaning with non-adapted products.
- REGULAR PREVENTIVE AND ANNUAL MAINTENANCE IS NOT CARRIED OUT in accordance with the annex B of the EN ISO 10535 standard, and the advices and instructions presented in this manual.

End of life

The end of life disposal of the device must be done respecting the recycling circuit and local relevant regulation.

For waste management that respects the environment, please kindly contact the sales manager in your area.

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ANNEXE 1

Reference	Description	Class	UDI-DI
S8511000	Polyester Comfort sling without head support – XS	Class I	3664844002667
S8511100	Polyester Comfort sling without head support – S	Class I	3664844002650
S8511200	Polyester Comfort sling without head support – M	Class I	3664844002643
S8511300	Polyester Comfort sling without head support – L	Class I	3664844002636
S8511400	Polyester Comfort sling without head support – XL	Class I	3664844002629
S8511500	Polyester Comfort sling without head support – XXL	Class I	3664844002612
S8513000	Polyester Comfort sling with Comfort head support – XS	Class I	3664844002568
S8513100	Polyester Comfort sling with Comfort head support – S	Class I	3664844002551
S8513200	Polyester Comfort sling with Comfort head support – M	Class I	3664844002544
S8513300	Polyester Comfort sling with Comfort head support – L	Class I	3664844002537
S8513400	Polyester Comfort sling with Comfort head support – XL	Class I	3664844002520
S8513500	Polyester Comfort sling with Comfort head support – XXL	Class I	3664844002513
S8521100	Polyester Hammock sling – S	Class I	3664844002384
S8521200	Polyester Hammock sling – M	Class I	3664844002377
S8521300	Polyester Hammock sling – L	Class I	3664844002360
S8533000	Vinyl sling Comfort with Stiffened head support – XS	Class I	3664844002445
S8533100	Vinyl sling Comfort with Stiffened head support – S	Class I	3664844002438
S8533200	Vinyl sling Comfort with Stiffened head support – M	Class I	3664844002421
S8533300	Vinyl sling Comfort with Stiffened head support – L	Class I	3664844002414
S8533400	Vinyl sling Comfort with Stiffened head support – XL	Class I	3664844002407
S8533500	Vinyl sling Comfort with Stiffened head support – XXL	Class I	3664844002391
S8552200	Polyester Toilet sling for electric suspension without head support – M	Class I	3664844002117
S8552300	Polyester Toilet sling for electric suspension without head support – L	Class I	3664844002100
S8552400	Polyester Toilet sling for electric suspension without head support – XL	Class I	3664844002094
S8553200	Polyester Comfort sling for electric suspension with head support – M	Class I	3664844002087
S8553300	Polyester Comfort sling for electric suspension with head support – L	Class I	3664844002070
S8553400	Polyester Comfort sling for electric suspension with head support – XL	Class I	3664844002063
S8554200	Polyester net bath sling for electric suspension with head support – M	Class I	3664844002056
S8554300	Polyester net bath sling for electric suspension with head support – L	Class I	3664844002049

DOQA214 -EU Declaration of Conformity of the slings - 20221012

2 / 5

S8554400	Polyester net bath sling for electric suspension with head support – XL	Class I	3664844002032
S8555200	Polyester All day sling for electric suspension – M	Class I	3664844002148
S8555300	Polyester All day sling for electric suspension – L	Class I	3664844002131
S8555400	Polyester All day sling for electric suspension – XL	Class I	3664844002124
S8580200	Polyester Amputee sling -Post operative sling – M	Class I	3664844002025
S8580300	Polyester Amputee sling -Post operative sling – L	Class I	3664844002018
S8590100	Polyester quick active Self Adjusting Sling with Abdominal Belt S	Class I	3664844001929
S8590200	Polyester quick active Self Adjusting Sling with Abdominal Belt M	Class I	3664844001912
S8590300	Polyester quick active Self Adjusting Sling with Abdominal Belt L	Class I	3664844001905
S8590400	Polyester quick active Self Adjusting Sling with Abdominal Belt XL	Class I	3664844001899
S8591100	Polyester active transfer sling Cocoon with Abdominal Belt– S	Class I	3664844001967
S8591200	Polyester active transfer sling Cocoon with Abdominal Belt– M	Class I	3664844001950
S8591300	Polyester active transfer sling Cocoon with Abdominal Belt– L	Class I	3664844001943
S8591400	Polyester active transfer sling Cocoon with Abdominal Belt– XL	Class I	3664844001936
S8592100	Polyurethan active Comfort sling with Abdominal Belt– S	Class I	3664844001882
S8592200	Polyurethan active Comfort sling with Abdominal Belt– M	Class I	3664844001875
S8592300	Polyurethan active Comfort sling with Abdominal Belt– L	Class I	3664844001868
S8592400	Polyurethan active Comfort sling with Abdominal Belt– XL	Class I	3664844001851
S8593100	Polyurethan active Comfort sling with Abdominal Belt and leg support– S	Class I	3664844003206
S8593200	Polyurethan active Comfort sling with Abdominal Belt and leg support– M	Class I	3664844003213
S8593300	Polyurethan active Comfort sling with Abdominal Belt and leg support– L	Class I	3664844003220
S8593400	Polyurethan active Comfort sling with Abdominal Belt and leg support– XL	Class I	3664844003237
S8594100	Polyester active transfer sling for home with Abdominal Belt - S	Class I	3664844002001
S8594200	Polyester active transfer sling for home with Abdominal Belt - M	Class I	3664844001998
S8594300	Polyester active transfer sling for home with Abdominal Belt - L	Class I	3664844001981
S8594400	Polyester active transfer sling for home with Abdominal Belt - XL	Class I	3664844001974
S8610100	Polyester All day sling without head support – S	Class I	3664844002711
S8610200	Polyester All day sling without head support – M	Class I	3664844002704
S8610300	Polyester All day sling without head support – L	Class I	3664844002698
S8610400	Polyester All day sling without head support – XL	Class I	3664844002681
S8610500	Polyester All day sling without head support – XXL	Class I	3664844003152

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S8311302	Intensive care stainless steel stretcher 4 vinyl slings	Class I	3664844001806
S8311301	Intensive care stainless steel stretcher one-part sling	Class I	3664844001790
S1921002	Radiolucent flat stretcher in 2 parts for 240 kg	Class I	3664844001783
S1921001	Radiolucent flat stretcher in 2 parts for 300 kg	Class I	3664844001776

Contact information

The information and advice provided in this user manual are also intended to inform you of the importance of paying attention to safety measures and the necessity of regular maintenance.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

This is a matter of responsibility for your establishment, and a question of safety for your patients and staff.

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S8612100	Polyester home transfer sling with head support – S	Class I	3664844002605
S8612200	Polyester home transfer sling with head support – M	Class I	3664844002599
S8612300	Polyester home transfer sling with head support – L	Class I	3664844002582
S8612400	Polyester home transfer sling with head support – XL	Class I	3664844002575
S8614000	Polyester Soft sling with head support, Comfort leg support – XS	Class I	3664844002506
S8614100	Polyester Soft sling with head support, Comfort leg support – S	Class I	3664844002490
S8614200	Polyester Soft sling with head support, Comfort leg support – M	Class I	3664844002483
S8614300	Polyester Soft sling with head support, Comfort leg support – L	Class I	3664844002476
S8614400	Polyester Soft sling with head support, Comfort leg support – XL	Class I	3664844002469
S8614500	Polyester Soft sling with head support, Comfort leg support – XXL	Class I	3664844002452
S8620100	Polyester net bath sling without head support – S	Class I	3664844002353
S8620200	Polyester net bath sling without head support – M	Class I	3664844002346
S8620300	Polyester net bath sling without head support – L	Class I	3664844002339
S8620400	Polyester net bath sling without head support – XL	Class I	3664844002322
S8623100	Polyester net bath sling with head support – S	Class I	3664844002308
S8623200	Polyester net bath sling with head support – M	Class I	3664844002292
S8623300	Polyester net bath sling with head support – L	Class I	3664844002285
S8623400	Polyester net bath sling with head support – XL	Class I	3664844002278
S8642000	Polyester Toilet slings without head support – XS	Class I	3664844002261
S8642100	Polyester Toilet slings without head support – S	Class I	3664844002254
S8642200	Polyester Toilet slings without head support – M	Class I	3664844002247
S8642300	Polyester Toilet slings without head support – L	Class I	3664844002230
S8642400	Polyester Toilet slings without head support – XL	Class I	3664844002223
S8642500	Polyester Toilet slings without head support – XXL	Class I	3664844002216
S8649000	Polyester removable head support for Toilet sling – XS	Class I	3664844002209
S8649100	Polyester removable head support for Toilet sling – S	Class I	3664844002193
S8649200	Polyester removable head support for Toilet sling – M	Class I	3664844002186
S8649300	Polyester removable head support for Toilet sling – L	Class I	3664844002179
S8649400	Polyester removable head support for Toilet sling – XL	Class I	3664844002162
S8649500	Polyester removable head support for Toilet sling – XXL	Class I	3664844002155

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